MARCOLIN

BLUE BLOCK

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Montature da vista

EN BLUE BLOCK MEDICAL DEVICE INFORMATION BOOKLET
INSTRUCTIONS FOR USE

INSTRUCTIONS AND BENEFITS Blue Block is a medical device, intended to prevent or relieve eye strain caused by exposure to blue light from artificial sources, such as personal computer, tablet or smartphone screens, or similar electronic devices. Exposure to artificial blue light - of the kind emitted by the screens of many electronic devices, including computers, tablets and smartphones - can cause eye strain symptoms, especially if prolonged. Blue Block may contribute to the reduction or prevention of the onset of such symptoms. Blue Block may be used by adults and teenagers. The device may be used together with contact lenses. The frame used for the manufacture of Blue Block devices complies with the requirements set out in standard EN ISO 12870:2018.

INSTRUCTIONS FOR USE Blue Block is used in the same way as regular glasses. Ensure the lenses are thoroughly clean before using the device. Should the lenses or frames appear to be dirty, the device can be washed with mild soap and warm water. Do not use solvents (e.g., alcohol and acetone) or aggressive detergents which may alter the features of the glasses. Always place the device back in its case after use.

CONTRAINDICATIONS Blue Block is not a pair of sunglasses and should not be used as personal protective equipment to protect the eyes against the sun. Blue Block is not to be used by children and infants.

SIDE-EFFECTS, WARNINGS AND PRECAUTIONS Use of the device may lead to changes in the sleep cycle. Depending on the individual and on the lighting conditions in which the individual uses the device, such changes may be either positive (finding it easier to fall asleep) or negative (difficulty in falling asleep). Should the device have a negative impact, if the other conditions for use are met, halt the use of the device. In low lighting, use of the device may reduce visual performance. Adverse contact reactions to materials and treatments used for the frames in particularly sensitive or allergic subjects cannot be ruled out. Should the lenses be scratched to the extent that it impacts visibility, the device should no longer be used and should be replaced; we recommend the use of original replacement spare parts.

STORAGE AND CLEANING Store Blue Block in its case at a temperature between -10°C and +35°C. Keep away from direct sunlight and avoid exposure to high temperatures, such as the heat in a car without conditioning (e.g., car dashboard), because high temperatures may alter the features of Blue Block. In case of contact with agents such as salt water (seawater), swimming pool water, chemical products such as make-up, hairspray, sun lotions, and insect repellents, we recommend that you immediately remove such agents using a soft damp cloth. Important information. Eye strain may be caused by a range of factors beyond exposure to the blue light emitted from

electronic device screens, such as uncorrected refractive errors, the distance of the screen from the eyes, as well as conditions in the surrounding environment (lighting, reflections on the screen etc...). The device is not intended as a cure for eye strain, although it may contribute to the reduction of the symptoms. Should the strain persist, it is important to take these factors into consideration and consider consulting with your doctor or a specialist.

MANUFACTURER Marcolin S.P.A. Zona Industriale Villanova 4 - 32013 - Longarone (Belluno) – Italy

REPORTING ADVERSE EFFECTS

Should you experience serious repercussions related to use of the device, you should immediately report it to Marcolin S.p.A. Zona Industriale Villanova 4 - 32013 - Longarone (Belluno) - Italy or by e-mail to servizioclienti@marcolin.com and to the relevant authorities of the European Union member state in which the user resides.

DISPOSAL METHOD The Blue Block device may be considered as household waste; dispose of the device and its packaging in accordance with the current regulations. Do not discard the Blue Block device and its case in the environment.

MARKING The model identification mark is on the inside of the temple of the glasses. The date of manufacture is indicated in the format "MM/YY" on a label placed on the individual packaging of the glasses.

The information provided by the manufacturer and the EU declaration of conformity for the product are available at the following address www.marcolin.com

SERIAL NUMBER The serial number is indicated on the inside of the temple of the glasses and on a label placed on the individual packaging after the symbol "SN".

LABEL/INDIVIDUAL PACKAGING SYMBOLS

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Donawa Lifescience Ltd
Marcolin (UK) Limited